



Denbur, Inc.

Quality Dental Manufacturing
Established 1979

[Home](#)

[Dental](#)

[Beauty](#)

[Meetings](#)

[Newsletters](#)

[About Us](#)

[Contact Us](#)

How to use and setup up your Single-Take Dispenser


Step 1:

Locate ridged track on Single-Take dispenser.



Step 2:

Locate the track on both bottom sides of the POD. The recyclable plastic sheet called the tongue will currently be lined up in the track. This part will be automatically removed in the following steps.

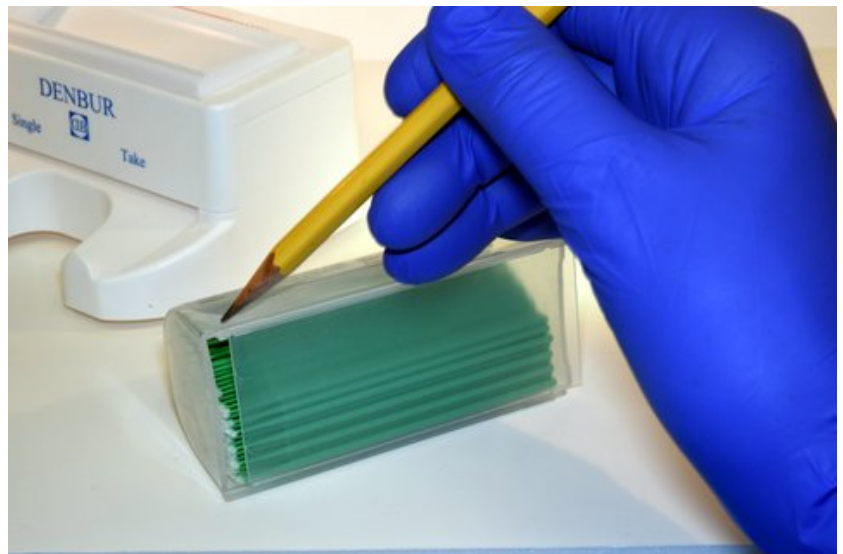

Step 3:

With the Single-Take dispenser facing you, from the right side line up track of the POD with the ridged track of the Single-Take dispenser. Load following the direction of the red arrow.

Step 4:

If properly loaded, the plastic tongue underneath the POD will begin to slide back away to release the applicators into the Single-Take dispenser.



Step 5:

Slide the POD all the way to the end until you hear a click. Once you hear the click your POD is correctly lined up with the Single-Take Dispenser.



Step 6:

Press down gently on top of the POD until it comes to a stop. One applicator will be dispensed and roll to the front of the Single-Take dispenser for use.

Step 7:

Remove applicator for use or dispense up to 5 applicators to keep ready for use.

